

INSTRUCTION FOR USE

Three Dimension Mask Grapefruit Fragrance

SIZE GUIDE

MS Size: 90 mm × 160 mm

FEATURES

Three Dimension Mask
Blocks 99%* Virus and Bacteria Droplets
Also for PM2.5 protection

Grapefruit Fragrance

Make up Smudge Proof

Nose fitting bar to ensure a perfect fit to the nose shape

Ample space around the mouth

Special coating prevents make-up from smearing

Aroma microcapsules enclosed within the mask release long lasting fragrance

Extra soft and highly stretchable ear loops

Double trapping filters cut 99%* *Tested by Nelson Laboratories, USA
PFE (Filtration efficiency of 0.0001mm fine particles),
VFE (Filtration efficiency for virus-containing droplets),
BFE (Filtration efficiency for cough and sneeze droplets)

Dermatologically tested: This does not mean that no one may have skin irritation.

INDICATION

Protection from virus, pollen and PM 2.5

INSTRUCTION FOR USE

1. Tap the mask gently to activate the aroma microcapsules. The stronger you tap, the stronger the fragrance.
2. Stretch the pleats lengthwise and spread the mask before wearing.
3. Adjust the nose fitting bar on the upper part of the mask along with the nose.

INSTRUCTION FOR USE

The side with flaps should be in contact with your face.

1 Tap the mask gently to activate the aroma microcapsules. The stronger you tap, the stronger the fragrance.

2 Stretch the pleats lengthwise and spread the mask before wearing.

3 Adjust the nose fitting bar on the upper part of the mask along with the nose.

Bend the nose fitting bar in advance.

CAUTION

- Read the above instructions carefully for proper use.
- The “smear-free” effect may not be fully achieved depending on the type, quantity, or condition of the cosmetics you wear.
- Your makeup may smudge if you hold down the nose fitting bar on the nose repeatedly after wearing the mask.

WARNING

- Infants and people with respiratory system abnormalities should not use this product.
- Stop using if you experience nausea or difficulty to breathe due to the odor of product materials.
- Do not use if you have abnormality on skin.
- If you experience itchiness or rash, stop using immediately and consult a physician.
- Do not use the mask near fire.
- Use the mask as soon as you open the individual package.
- This product is not intended for protection from toxic gas and hazardous dust.
- This is a disposable mask. Do not wash and reuse. For hygiene reasons, it is desirable that you use one piece each day and replace with a new one whenever you find the one you are using dirty.
- Nose fitting bar is attached on the upper part of the mask; use with caution.
- Your glasses may fog up depending on the environment or the user; use with caution when driving.
- Keep out of reach of infants.
- Avoid high temperature and humidity.

PACKAGING

5 pieces/pack

PRODUCT OWNER

Kowa Company, Ltd.

4-14, Nihonbashi-honcho 3-chome, Chuo-ku, Tokyo, JAPAN

MANUFACTURER

SAN-M PACKAGE CO., LTD.

1086-1 OJIRO, SHIMADA-CITY, SHIZUOKA, JAPAN