

INSTRUCTION FOR USE

Three Dimension Mask MD

SIZE GUIDE

S Size: Width 145 mm

M Size: Width 175 mm

FEATURES

The Three Dimension Mask MD is a face mask with the following features:

<Highly functional>

Extra-fine trapping filter

PFE (Filtration efficiency of 0.0001mm tiny particles),

VFE (Filtration efficiency for virus-containing droplets),

BFE (Filtration efficiency for bacteria-containing droplets)

* Filtration efficiency testing performed by Nelson Laboratories, LLC, USA and Eurofins Textile Testing Spain (PFE: ASTM F2299, BFE: ASTM F2101-19 and EN14683:2019, Annex B, VFE: ASTM F2101)

Natural catechin*extract antibacterial filter

Suppresses bacterial growth on filter surface

*Antibacterial activity tested by Nissenken Quality Evaluation Center (JIS L1902:2015, tested bacteria: E. coil, S. aureus)

*Catechin is a main ingredient of green tea which is known for its antibacterial and antiviral effects.

Prevents gaps around the nose and the cheeks

The shape retaining nose fitting bar and the side fitting form prevent gaps

<Comfortable>

Prolonged wearing comfort

The extra soft and wide ear-loops minimize ear pain after prolonged wearing

Skin friendly nonwoven fabric

Soft fabric that is gentle on your skin

Loose fitting around the mouth

Allows you to breathe and talk easily

INDICATION

For protection from virus, bacterial droplets, pollens and tiny particles.

INSTRUCTION FOR USE

1. The sign with “inside” should contact the face.
2. Unfold the pleats and open the mask.
3. Adjust the nose fitting bar on the upper part of the mask along with the nose.



PRECAUTIONS FOR USE

- After wearing, do not touch the outer surface of the mask.
- Do not return a used mask into the package.
- This is a disposable mask. Do not wash and reuse. Replace with a new mask when the one you are using is dirty.
- Take out of the package only as many pieces as you use.
- Nose fitting bar is attached on the upper part of the mask.
- Do not use for people with skin abnormalities, such as atopic dermatitis, acne vulgaris, rough skin, etc.
- If you experience itchiness or rash, stop using immediately and consult a physician or a pharmacist.
- Stop using if you experience nausea or difficulty to breathe.
- Avoid high temperature and humidity.

CONTRAINDICATIONS

- Infants and people with respiratory system abnormalities, such as asthma, etc. should not use this product.
- Your glasses may fog up depending on the environment or the user; use with caution.
- Keep out of reach of infants.

PACKAGING

5 pieces / plastic pack

30 pieces / box

PRODUCT OWNER

Kowa Company, Ltd.

4-14, Nihonbashi-honcho 3-chome, Chuo-ku, Tokyo, JAPAN