

# INSTRUCTION FOR USE

## VANTELIN SUPPORT KNEE


### SIZE GUIDE

S: 12-13 in / 31-34 cm

M: 13-14.5 in / 34-37 cm

L: 14.5-16 in / 37-40 cm

XL: 16-17 in / 40-43 cm

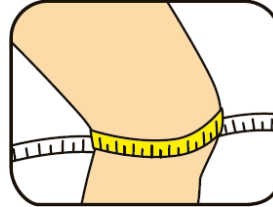

Measure around knee as shown. If two sizes apply, choose the larger.

### FEATURES


|   |  |
|---|--|
|   | <p><b>U-Shaped Taping Structure</b></p> <p>U-shaped taping structure supports the kneecap to reduce stress to the patellar tendon when extending the knee.</p> |
|  | <p><b>Slit Knit Structure</b></p> <p>Breathable front and back mesh panels provide ventilation and reduce bunching behind the knee.</p>                        |
|  | <p><b>Anchor Structure</b></p> <p>Seamless top and bottom cuffs prevent slipping of the support and stabilize the U-shaped taping structure.</p>               |

### INDICATION

Used for daily joint discomfort.

## INSTRUCTION FOR USE

1. Slide leg into knee support with the bottom of the “V” pointing down.
  2. Position the U-shaped taping structure below and on either side of your knee.  
Smooth the brace so there is no bunching when you stretch your knee.
- Always wear the support directly on skin.
  - If the support is not positioned correctly, it may not function as intended.
- \* Functions obtained by wearing the support may vary depending on each person.

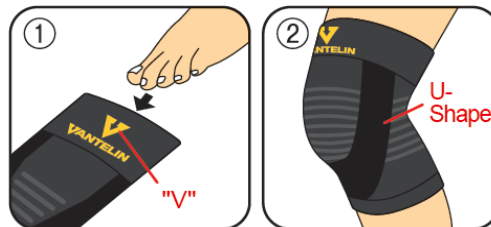

## WARNING

- Do not use if you have: (1) abnormalities including swelling and strong pain due to sprain, fracture, or ligament injury etc., (2) peripheral vascular disorder, (3) abnormality such as injury, swelling, rash at site where support is to be worn, or (4) chronic dermatitis / allergy to chemical textile or rubber. If you experience symptoms such as rash / redness, itchiness, eruption, congestion, abnormal pain, or swelling, stop using immediately and consult your healthcare professional.
- Use only as directed.
- Use correct size.
- Do not use when sleeping.

## CARE INSTRUCTIONS (Use laundry net)

1. Wash separately below 30°C.
2. Do not bleach.
3. Do not tumble dry.
4. Hang and air-dry. Avoid direct sunlight.
5. Iron at low temperature. Do not steam iron.
6. Do not dry clean.

## PRECAUTIONS CONCERNING STORAGE AND HANDLING

1. Do not place the product close to fire and heat.
2. Keep out of reach of children.
3. There is a risk of the product tearing if worn in a wrong way or strong force is applied to part of the product.

**PACKAGING**

Box of 1 unit

**PRODUCT OWNER**

Kowa Company, Ltd.

4-14, Nihonbashi-honcho 3-chome, Chuo-ku, Tokyo, JAPAN

**MANUFACTURING SITE**

IIDA HOSIERY CO., LTD.

1619, Yamaguchi, Kouhoku-machi, Kishima-gun, Saga-Ken 849-0501 Japan