


INSTRUCTION FOR USE

VANTELIN SUPPORT COOL FIT Knee (Light Blue)

SIZE GUIDE:




M: 13in-14.5in / 34cm-37cm

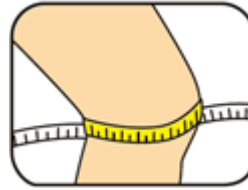L: 14.5in-16in / 37cm-40cm



Measure around knee as shown. If two sizes apply, chose the larger.

FEATURES

	<p>U-shaped Taping Structure</p> <p>U-shaped taping structure supports the kneecap to reduce stress to the patellar tendon when extending the knee</p>
	<p>Slit Knit structure</p> <p>Breathable front and back mesh panels provide ventilation and reduce bunching behind the knee</p>
	<p>Anchor structure</p> <p>Seamless top and bottom cuffs prevent slipping of the support and stabilize the U-shaped taping structure</p>


INDICATION

Used for daily joint discomfort.

INSTRUCTION FOR USE

1. Slide leg into knee support with the bottom of the “V” pointing down.
2. Position the U-shaped taping structure below and on either side of your knee.

3. Smooth the brace so there is no bunching when you bend your knee. Always wear the support directly on skin. If the support is not positioned correctly, it may not function as intended.

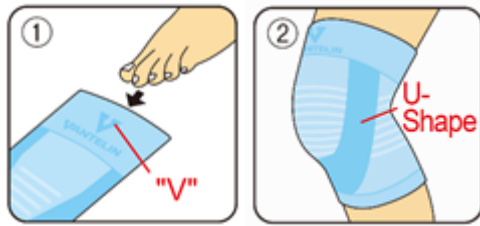

WARNING

- Do not use if you have: (1) a sprain, fracture, or ligament injury, (2) peripheral vascular disorder, (3) abnormality such as injury, blotchiness, rash at site where support is to be worn, or (4) chronic dermatitis/allergy to chemical textile or rubber.

If you experience any of these, stop using immediately and consult your healthcare professional.

- Use only as directed.
- Use correct size.
- Do not use when sleeping.

CARE INSTRUCTIONS

1. Hand wash separately below 40°C.
2. Do not bleach.
3. Do not tumble dry.
4. Hang and air-dry. Avoid direct sunlight.
5. Iron at low temperature. Do not steam iron.
6. Do not dry clean.

Precautions concerning storage and handling

1. Do not put the product close to the fire and heat.
2. Keep out of reach of children.
3. There is a risk of the product breaking when wearing in a wrong way or applying a strong force partially.

PACKAGING

Box of 1 unit.

PRODUCT OWNER

KOWA COMPANY, LTD

4-14, 3-chome, Nihonbashi-Honcho, Chuo-ku, Tokyo 103-8433, Japan

MANUFACTURING SITE

IIDA HOSIERY CO., LTD.

1619, Yamaguchi, Kouhoku-machi, Kishima-gun, Saga-Ken 849-0501 Japan.