

INSTRUCTION FOR USE

VANTELIN SUPPORT ANKLE

SIZE GUIDE (Shoe Size)

S: Women: 4.5-6.5 in / 22-24 cm

Men: 4-6 in / 22-24 cm

M: Women: 6.5-8.5 in / 24-26 cm

Men: 6-8 in / 24-26 cm

L: Women: 8.5-10.5 in / 26-28 cm

Men: 8-10 in / 26-28 cm

If two sizes apply, choose the larger.

FEATURES


 A top-down view of a right foot wearing a black VanTELIN support sock. Red dashed lines illustrate a figure-eight taping pattern that starts at the heel, goes up the inner side of the ankle, across the top of the foot, down the outer side of the ankle, and back across the top of the foot.	<p>Figure-Eight Taping Structure</p> <p>The knitted mesh figure-eight wraps around the ankle and foot to provide extra joint support when running, walking and performing daily activities.</p>
 A side view of a right foot wearing the black support sock. A red dashed line highlights a curved support structure that follows the arch of the foot.	<p>Arch Support Taping Structure</p> <p>The arch support taping structure raises the arch to enhance its cushioning function, thereby reducing fatigue.</p>
 A side view of a right foot wearing the black support sock. Red dashed lines highlight the top cuff at the ankle and the bottom cuff at the heel, which serve as anchors for the support structure.	<p>Anchor Structure</p> <p>Seamless top and bottom cuffs prevent slipping of the support and stabilize the figure-eight taping structure.</p>

INDICATION

Used for daily joint discomfort.

INSTRUCTION FOR USE

1. Slide foot into ankle support with the bottom of the “V” pointing down.
 2. Fit the support to the ankle and adjust without slack.
 - Always wear the support directly on skin.
 - If the support is not positioned correctly, it may not function as intended.
- * Functions obtained by wearing the support may vary depending on each person.

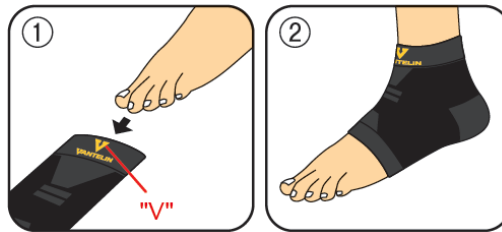

WARNING

- Do not use if you have: (1) abnormalities including swelling and strong pain due to sprain, fracture, or ligament injury etc., (2) peripheral vascular disorder, (3) abnormality such as injury, swelling, rash at site where support is to be worn, or (4) chronic dermatitis / allergy to chemical textile or rubber. If you experience symptoms such as rash / redness, itchiness, eruption, congestion, abnormal pain, or swelling, stop using immediately and consult your healthcare professional.
- Use only as directed.
- Use correct size.
- Do not use when sleeping.

CARE INSTRUCTIONS (Use laundry net)

1. Wash separately below 30°C.
2. Do not bleach.
3. Do not tumble dry.
4. Hang and air-dry. Avoid direct sunlight.
5. Iron at low temperature. Do not steam iron.
6. Do not dry clean.

PRECAUTIONS CONCERNING STORAGE AND HANDLING

1. Do not place the product close to fire and heat.
2. Keep out of reach of children.
3. There is a risk of the product tearing if worn in a wrong way or strong force is applied to part of the product.

PACKAGING

Box of 1 unit

PRODUCT OWNER

Kowa Company, Ltd.

4-14, Nihonbashi-honcho 3-chome, Chuo-ku, Tokyo, JAPAN

MANUFACTURING SITE

IIDA HOSIERY CO., LTD.

1619, Yamaguchi, Kouhoku-machi, Kishima-gun, Saga-Ken 849-0501 Japan