

INSTRUCTION FOR USE


VANTELIN SUPPORT BACK

SIZE GUIDE

M : 26-34 in / 65-85 cm

L : 32-39 in / 80-100 cm

XL : 37-45 in / 95-115 cm

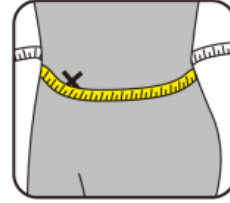

Measure your waistline at your navel as shown. If two sizes apply, choose the larger.

FEATURES

A grayscale illustration of a human torso from the waist up. A red outline highlights the lower back and hip area. Overlaid on this area is a yellow cross-net pattern, representing the support taping.	<p>Restricts Backward Bending</p> <p>Built-in Cross-Net Taping provides wide coverage of the lower back. It limits excessive “back-and-forth motion” and “twist” which cause pain.</p>
A grayscale illustration of the lower abdomen and lower back. Red arrows point inward from the sides towards the spine, indicating the firm support provided by the device. A dashed line shows the natural curvature of the spine.	<p>Helps Improve Posture</p> <p>Ergonomically designed support holds the lower abdomen and lower back firmly in place to help establish a healthy posture.</p>
	<p>Enhanced Breathability</p> <p>Use of thin breathable fabric improves ventilation to help you stay cool and dry.</p>

INDICATION

Used for lower back discomfort.

INSTRUCTION FOR USE: Make sure to wear the brace over your underwear.

1. Hold the brace with the tag on the right inner side (arrows should be pointing up), and pull it with both hands to make both sides equal in length.
2. Pull the left side obliquely downward (as a guide, approx. 5 cm/2 in below the navel). * Do not wear it above the navel.
3. Fix the left hand position and pull the right side obliquely downward to tape the hook and loop fastener.



WARNING

- Do not use if you have: (1) abnormalities such as swelling and strong pain on the injured area, including bruises and fractures, (2) peripheral vascular disorder, (3) abnormality such as injury, swelling, rash at site where support is to be worn, or (4) chronic dermatitis / allergy to chemical textile or rubber. If you experience symptoms such as rash / redness, itchiness, eruption, congestion, abnormal pain, or swelling, stop using immediately and consult your healthcare professional.
- Do not use if you are or may become pregnant.
- Use correct size.
- Do not use when sleeping.
- Do not over-tighten.
- Do not wear brace for extended periods of time.
- Be cautious on clothes getting damaged by the hook-and-loop faster.
- Do not scratch with your nails or pull hard.

* If the support is not positioned correctly, it may not function as intended.

* Functions obtained by wearing the support may vary depending on each person.

CARE INSTRUCTIONS (Close fasteners prior to washing)

1. Hand wash separately below 40°C. Do not wring.
2. Do not bleach.
3. Do not tumble dry.
4. Hang and air-dry. Avoid direct sunlight.
5. Do not iron.
6. Do not dry clean.
7. Gentle wet cleaning.

PRECAUTIONS CONCERNING STORAGE AND HANDLING

1. Do not place the product close to fire and heat.
2. Keep out of reach of children.
3. Store the hook and loop fastener in a fastened state so that lint or the like does not adhere.

PACKAGING

Box of 1 unit.

PRODUCT OWNER

Kowa Company, Ltd.

4-14, Nihonbashi-honcho 3-chome, Chuo-ku, Tokyo, JAPAN

MANUFACTURING SITE

D-M CHAIN Cooperative

5-321 Osaki, Kahoku-shi, Ishikawa 929-1127 Japan