

INSTRUCTION FOR USE


VANTELIN SUPPORT ELBOW

SIZE GUIDE

S: 7.5-9 in / 19-22 cm




M: 9-10 in / 22-25 cm

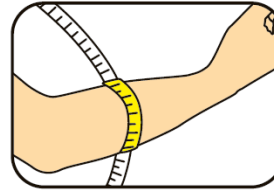L: 10-11 in / 25-28 cm



Measure around forearm as shown. If two sizes apply, choose the larger.

FEATURES

 <p>Inside Elbow</p>	<p>X-Shaped Taping Structure</p> <p>X-shaped taping structure on the inside of the elbow limits excessive movement and reduces discomfort when lifting heavy objects.</p>
 <p>Outside Elbow</p>	<p>Ventilated Mesh Panels</p> <p>The knitted mesh structure located on the outside of elbow allows the support to adapt to the elbow's movement.</p>
 <p>Anchor Structure</p>	<p>Anchor Structure</p> <p>Seamless top and bottom cuffs prevent slipping of the support and stabilize the X-shaped taping structure.</p>

INDICATION

Used for daily joint discomfort.


INSTRUCTION FOR USE

1. Slide arm into elbow support with the bottom of the “V” pointing down.
2. Position the knitted mesh X-shape on the inside of your elbow, with the center of the X aligned to the crease of your elbow.

Smooth the brace so there is no bunching when slightly bend your elbow.

- Always wear the support directly on skin.
- If the support is not positioned correctly, it may not function as intended.

* Functions obtained by wearing the support may vary depending on each person.

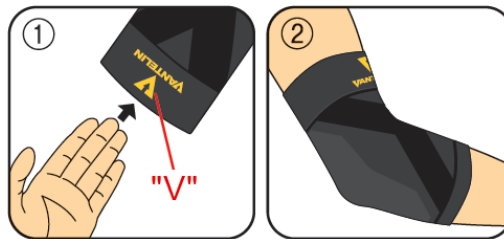

WARNING

- Do not use if you have: (1) abnormalities including swelling and strong pain due to sprain, fracture, or ligament injury etc., (2) peripheral vascular disorder, (3) abnormality such as injury, swelling, rash at site where support is to be worn, or (4) chronic dermatitis / allergy to chemical textile or rubber. If you experience symptoms such as rash / redness, itchiness, eruption, congestion, abnormal pain, or swelling, stop using immediately and consult your healthcare professional.
- Use only as directed.
- Use correct size.
- Do not use when sleeping.

CARE INSTRUCTIONS (Use laundry net)

1. Wash separately below 30°C.
2. Do not bleach.
3. Do not tumble dry.
4. Hang and air-dry. Avoid direct sunlight.
5. Iron at low temperature. Do not steam iron.
6. Do not dry clean.

PRECAUTIONS CONCERNING STORAGE AND HANDLING

1. Do not place the product close to fire and heat.
2. Keep out of reach of children.
3. There is a risk of the product tearing if worn in a wrong way or strong force is applied to part of the product.

PACKAGING

Box of 1 unit

PRODUCT OWNER

Kowa Company, Ltd.

4-14, Nihonbashi-honcho 3-chome, Chuo-ku, Tokyo, JAPAN

MANUFACTURING SITE

IIDA HOSIERY CO., LTD.

1619, Yamaguchi, Kouhoku-machi, Kishima-gun, Saga-Ken 849-0501 Japan