

# INSTRUCTION FOR USE

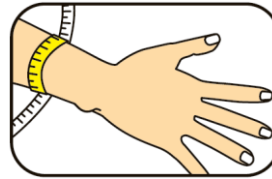
## VANTELIN SUPPORT WRIST

### SIZE GUIDE

S: 5.5-6 in / 13-15 cm

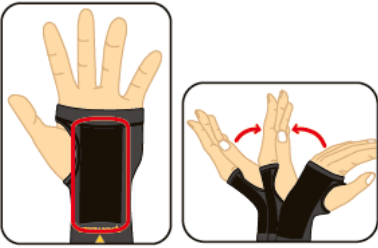


M: 6-6.5 in / 15-17 cm

L: 6.5-7.5 in / 17-19 cm



Measure around arm about 3 in / 7 cm above wrist. If two sizes apply, choose the larger.

### FEATURES

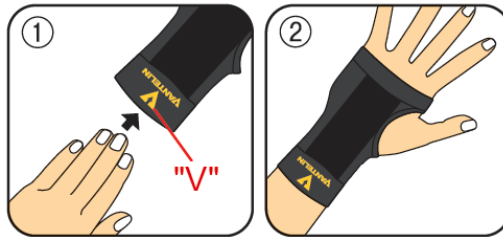
	<p><b>Sandwich-Taping Structure</b></p> <p>The sandwich-taping structure on the front and back of the wrist works like a splint to limit excessive wrist movement and reduce discomfort.</p>
	<p><b>Thumb-Lock Structure</b></p> <p>Maintains the correct position of the support while allowing free movement of the thumb.</p>
	<p><b>Anchor Structure</b></p> <p>Seamless top and bottom cuffs prevent the support from slipping and stabilize the taping structure.</p>

### INDICATION

Used for daily joint discomfort.

## INSTRUCTION FOR USE

1. Slide hand into wrist support with the bottom of the “V” pointing toward the fingers and the thumb protruding out the thumb hole.
  2. Smooth the brace and make sure the knitted mesh structures are straight and centered on the top and bottom of the wrist. Do not overstretch the support as it may put a strain on the back of your thumb.
    - Always wear the support directly on skin.
    - If the support is not positioned correctly, it may not function as intended.
- \* Functions obtained by wearing the support may vary depending on each person.



## WARNING

- Do not use if you have: (1) abnormalities including swelling and strong pain due to sprain, fracture, or ligament injury etc., (2) peripheral vascular disorder, (3) abnormality such as injury, swelling, rash at site where support is to be worn, or (4) chronic dermatitis / allergy to chemical textile or rubber. If you experience symptoms such as rash / redness, itchiness, eruption, congestion, abnormal pain, or swelling, stop using immediately and consult your healthcare professional.
- Use only as directed.
- Use correct size.
- Do not use when sleeping.

## CARE INSTRUCTIONS (Use laundry net)

1. Wash separately below 30°C.
2. Do not bleach.
3. Do not tumble dry.
4. Hang and air-dry. Avoid direct sunlight.
5. Iron at low temperature. Do not steam iron.
6. Do not dry clean.

## PRECAUTIONS CONCERNING STORAGE AND HANDLING

1. Do not place the product close to fire and heat.
2. Keep out of reach of children.
3. There is a risk of the product tearing if worn in a wrong way or strong force is applied to part of the product.

## PACKAGING

Box of 1 unit

**PRODUCT OWNER**

Kowa Company, Ltd.

4-14, Nihonbashi-honcho 3-chome, Chuo-ku, Tokyo, JAPAN

**MANUFACTURING SITE**

IIDA HOSIERY CO., LTD.

1619, Yamaguchi, Kouhoku-machi, Kishima-gun, Saga-Ken 849-0501 Japan