

INSTRUCTION FOR USE

VANTELIN SUPPORT COOL FIT Ankle (Light Blue)

SIZE GUIDE (Shoe Size)

M: Women : 6.5in~8.5in / 24cm~26cm




Men : 6in~8in / 24cm~26cm

L: Women : 8.5in~10.5in / 26cm~28cm

Men : 8in~10in / 26cm~28cm

If two sizes apply, chose the larger.

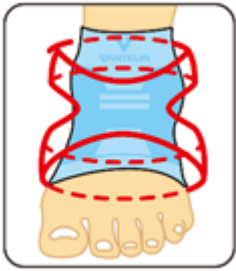FEATURES

 A diagram showing a light blue ankle support with a red dashed figure-eight pattern overlaid on the ankle and foot area, illustrating the taping structure.	<p>Figure-Eight Taping Structure</p> <p>The knitted mesh figure-eight wraps around the ankle and foot to provide extra joint support when running, walking and performing daily activities.</p>
 A diagram showing a light blue ankle support with a red dashed arch support structure overlaid on the arch of the foot.	<p>Arch Support Taping Structure</p> <p>The arch support taping structure raises the arch to enhance its cushioning function, thereby reducing fatigue.</p>
 A diagram showing a light blue ankle support with red dashed cuffs at the top and bottom, illustrating the anchor structure.	<p>Anchor Structure</p> <p>Seamless top and bottom cuffs prevent slipping of the support and stabilize the figure-eight taping structure.</p>


INDICATION

Used for daily joint discomfort.

INSTRUCTION FOR USE

1. Slide foot into ankle support with the bottom of the “V” pointing down.
2. Position the figure-eight taping structure around your ankle and arch of foot

- Smooth the brace so there is no bunching. Always wear the support directly on skin. If the support is not positioned correctly, it may not function as intended.

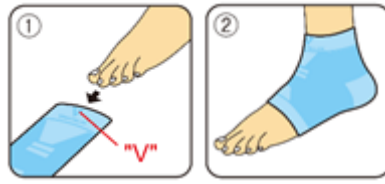

WARNING

- Do not use if you have: (1) a sprain, fracture, or ligament injury, (2) peripheral vascular disorder, (3) abnormality such as injury, blotchiness, rash at site where support is to be worn, or (4) chronic dermatitis/allergy to chemical textile or rubber.

If you experience any of these, stop using immediately and consult your healthcare professional.

- Use only as directed.
- Use correct size.
- Do not use when sleeping.

CARE INSTRUCTIONS

- Hand wash separately below 40°C.
- Do not bleach.
- Do not tumble dry.
- Hang and air-dry. Avoid direct sunlight.
- Iron at low temperature. Do not steam iron.
- Do not dry clean.

Precautions concerning storage and handling

- Do not put the product close to the fire and heat.
- Keep out of reach of children.
- There is a risk of the product breaking when wearing in a wrong way or applying a strong force partially.

PACKAGING

Box of 1 unit.

PRODUCT OWNER

KOWA COMPANY, LTD

4-14, 3-chome, Nihonbashi-Honcho, Chuo-ku, Tokyo 103-8433, Japan

MANUFACTURING SITE

IIDA HOSIERY CO., LTD.

1619, Yamaguchi, Kouhoku-machi, Kishima-gun, Saga-Ken 849-0501 Japan.